











REF	90213024
Pack Size	24 Cards

## Neonate Group Card

 +4°C Store at 4-25°C	 Manufacturer	 IVD In vitro Diagnostic Medical Device	 LOT Batch Number / Lot Number	 Expiry date	<b>NG</b>  SBioCAT™ Neonate Group Card
 Consult Instructions for use	 Date of Manufacture	 REF Catalogue Number	 This side up	 Keep Away from Sunlight	

### SUMMARY

Human red blood cell antigen can be divided into four groups A, B, AB and O depending on the presence or absence of corresponding antigens on the red blood cells. The Anti-A, Anti-B and Anti-AB reagents are used to detect the presence or absence of corresponding antigens on the red blood cells.

In Newborn's the A and B antigens are not completely developed, thus weaker reaction may be shown when compared to adults, when tested. In adults the antigens and respective antibodies are present, but in newborns the antibodies appear after 4 to 6 months of birth. Thus reverse grouping cannot be done on newborn samples, but it is possible after 2 to 4 years of birth.

Also human red blood cells are classified as Rho (D) positive or Rho(D) negative depending upon the presence or absence of Rho(D) antigen. The D antigen and weak D are fully developed at birth. If the mother is Rho(D) negative, it is very much important to determine the D antigen and weak D of the newborn's.

The Direct Antiglobulin Test is very important to detect the antibodies adsorbed to the red blood cells in-utero.

The determination of ABO, Rho(D) blood groups and the Direct Coombs Test can be made on newborn's blood cells using the SBioCAT™ Neonate Group Card. This technique will assist in prompt treatment of the Neonate.

### REAGENTS

SBioCAT™ Neonate Group Card contains six microtubes, pre-filled with gel in a suitable buffer containing specific Monoclonal Anti-A (Clone 11H5), Anti-B (Clone 6F9), Anti-AB (Clone ES-15+6F9), Anti-D (IgM) (VI-) (Clone P3x61 + NaTH119) antibodies and Neutral gel in appropriate microtube. The Anti Human Globulin reagent is a blend of polyclonal Anti Human IgG and Monoclonal Anti-C3d (Clone BRIC-8).

### STORAGE AND STABILITY

Store SBioCAT™ gel cards in an upright position at 4-25°C. Do not freeze. Avoid exposure of SBioCAT™ gel cards to direct sunlight or any heat source. The shelf life of SBioCAT™ gel cards is as per the expiry date mentioned on the label. Do not use beyond expiry date. Once the aluminium foil is removed from the microtube, it should be used immediately.

### ADDITIONAL REAGENTS AND MATERIALS REQUIRED

SBioCAT™ Diluent -2 LISS for preparation of red cell suspension. (Refer package insert before use). Gel card centrifuge (85g), Work station, Micropipette capable of delivering 5-50µl of specimen and Bottle top dispenser.

### PRINCIPLE

As the SBioCAT™ gel card containing red blood cells is centrifuged under specific conditions, the red blood cells possessing the corresponding antigen will agglutinate in presence of the specific antibody and will be trapped in the gel column. The red blood cells, which do not react are not trapped in the gel column and get settled at the bottom of the microtube. The reactions are then read and graded according to their reactivity pattern.

### SAMPLE COLLECTION

No special preparation of the patient is required prior to sample collection by approved techniques. For optimal results, freshly collected sample should be used. Anticoagulants like EDTA, CPD-A and Citrate can be used. Cord or heel prick samples may be used. (Cord blood samples must be washed at least 3 times with normal saline before testing).

### SAMPLE PREPARATION

Prepare a 0.8% red blood cell suspension in SBioCAT™ Diluent- 2 LISS as follows:

1. Bring the SBioCAT™ Diluent- 2 LISS to room temperature before testing.
2. Dispense 1.0 ml of SBioCAT™ Diluent- 2 LISS into a clean test tube.
3. Add 10µl of packed red cells and mix gently.
4. Red blood cell suspension so obtained should be used for testing.

### TEST PROCEDURE

1. Label the appropriate SBioCAT™ gel card with patient's name / identification number. Remove the aluminium foil carefully by pulling it backwards.
2. Pipette 50 µl of 0.8% red blood cell suspension to all the microtubes. Take care to ensure that the micropipette tip does not touch the reagent in the microtube.
3. Centrifuge the cards for 10 minutes in the gel card

- centrifuge.
4. Retrieve the card from centrifuge, read and record the results.

**INTERPRETATION OF RESULTS**

The control microtube (Ctrl) must be negative to validate the forward grouping results. If not, then repeat the test after washing the patient's red blood cells with warm saline.

**Positive reaction:** Agglutinated red blood cells forming a clear line on the surface of gel column or agglutinates dispersed in the gel column.

**Negative reaction:** Non agglutinated red blood cells settle at the bottom of the microtube forming a compact button.

The reaction strength may be recorded as follows:

Strength of reaction	Comments
4+	Agglutinated red blood cells form a line on the surface of the gel microtube.
3+	Most agglutinated red blood cells remain in the upper half of the gel microtube.
2+	Agglutinated red blood cells are observed throughout the length of the gel microtube. A small button of red blood cells may also be visible at the bottom of the gel microtube.
1+	Most agglutinated red blood cells remain in the lower half of the gel microtube. A button of cells may also be visible at the bottom of the gel microtube.
±	Most agglutinated red blood cells are in the lower third part of the gel microtube.
Negative	All the red blood cells pass through and form a compact button at the bottom of the gel microtube.
Mixed field agglutination	Agglutinated red blood cells form a line on the surface of the gel and non-agglutinated red blood cells form a compact button at the bottom of the gel microtube.
H	Hemolysis of red blood cells

Expected reactivity pattern for ABO grouping:

Anti- A	Anti- B	Anti- AB	Blood Group
± to 4+	Negative	± to 4+	A
Negative	± to 4+	± to 4+	B
± to 4+	± to 4+	± to 4+	AB
Negative	Negative	Negative	O

Expected reactivity pattern for Rho (D) typing:

Anti-D	Rho(D) Type
± to 4+	Rho (D) Positive
Negative	Rho (D) Negative

**NOTE:** Weak D/ Partial D type human red blood cells may give a weaker or negative reaction. Such cells should be retested for weak D confirmation with SBIOCAT™ Coombs Anti-IgG card.

**For Direct Antiglobulin Test:**

Negative reaction indicates absence of detectable IgG antibodies or Complement component C3d on the newborn's red blood cells.

Positive reaction indicates that newborn's red blood cells are sensitized with IgG or Complement component C3d.

**NOTE**

- In vitro diagnostic reagent for laboratory and professional use only. Not for medicinal use.
- The SBIOCAT™ gel cards contains sodium azide < 0.1% as preservative. Avoid contact with skin and mucosa. On disposal flush with large quantity of water.
- All SBIOCAT™ gel cards should be centrifuged for one complete cycle (10 minutes) in gel card centrifuge before use.
- Visually inspect the SBIOCAT™ gel cards before use.
- SBIOCAT™ gel cards having bubble(s) entrapped within the gel can be centrifuged for two complete cycles in gel card centrifuge to remove the bubble, if bubbles are not removed the card should not be used.
- SBIOCAT™ gel cards that exhibit any signs of drying (i.e. absence or reduced level of reagent buffer above the gel column), decreased volume of gel, cracked gel should not be used.
- SBIOCAT™ gel cards with damaged aluminium foil seal should not be used.
- Freezing of SBIOCAT™ gel cards or evaporation of gel or reagent buffer due to exposure to heat may lead to erroneous results.
- Fibrin or particulate matter if present in the sample may lead to erroneous results.
- Fibrin if present in the sample may trap red blood cells on the surface of the gel column presenting a pink line. RBCs should be washed with normal saline if not collected properly in an anticoagulant
- Use of red blood cells concentration/ volume and reagents other than those described may lead to erroneous results. Follow the instructions carefully.
- Aged or stored red blood cells may exhibit weaker reactivity than freshly collected cells.
- Old cell panels may give an unclear background with SBIOCAT™ gel cards.
- Do not use hemolysed samples.
- Extreme turbidity or discoloration may indicate microbial contamination or denaturation of protein due to thermal damage. Such SBIOCAT™ gel cards should be discarded.
- Contamination of reagents during usage may cause false positive or negative results.
- Red cell aggregation in the red cell suspension may interfere the passage.
- Aluminium foil seal of SBIOCAT™ gel cards should be removed gently and carefully by pulling the foil seal backwards to avoid contamination of reagents from

- one microtube to another.
19. Do not use lipemic and icteric samples.
  20. To avoid contamination always use fresh tips before dispensing into each microtube.
  21. Since A and B antigens are not fully developed at birth, weak reactions may occur with neonate samples and subgroups can not be identified.
  22. Some pathological conditions are reported as causing non-specific reactions in AHG procedures.
  23. SBIOCAT™ ContaVoid can be used to avoid contamination of reagents in microtubes while usage. For details refer pack insert of SBIOCAT™ ContaVoid (Cat. No.903300100).

#### REMARKS

1. Known positive and negative control should be tested as per Good Laboratory Practices.
2. SBIOCAT™ Red Cell Preserving Solution (Cat. No.90262020) can be used as red blood cell preservative solution for preservation of known cells.
3. The Anti-D does not detect the D VI variant.

#### PERFORMANCE

The performance study has been evaluated on 3275 blood samples (from donors, clinical and neonates) drawn in the recommended anticoagulants. The evaluation demonstrated 100% specificity of Anti-A, Anti-B and Anti-D reagent and 100% sensitivity of Anti-A and Anti-B reagents, 99.4% sensitivity of Anti-D reagent versus the expected results with common known phenotypes A<sub>1</sub>, A<sub>2</sub>, A, B, A<sub>2</sub>, B, B and O and with common known Rhesus phenotypes. Blood sample with weak D expression showed different reaction strength. The results of DAT showed complete concordance with the results of reference method.

#### BIBLIOGRAPHY

1. Human Blood Groups by Geoff Daniels, 2<sup>nd</sup> Edition, Blackwell Science, Oxford 2002.
2. HMSO, Guidelines for the Blood Transfusion Services, 2<sup>nd</sup> Edition, 1993.
3. D. Voak, New Developments in Blood Group Serology, Infusion Therapy Transfusion Medicine 1999;26:258-260.
4. Blood Transfusion in Clinical Medicine, P.L. Mollison, 10<sup>th</sup> Edition.
5. Data on file: Singapore Biosciences PTE Ltd.



Singapore   
Biosciences PTE Ltd.


11 Yishun Street 51, 304-23, The Criterion, Singapore 767971.

REF	90213024
NG	24 Cards
PACKAGE INSERT	1

LOT



G0DRUSS/357

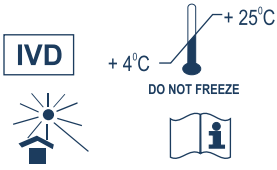

Singapore **SB**  
Biosciences PTE Ltd.



**SBioCAT**<sup>TM</sup>  
Column Agglutination Test

24 CARDS

Neonate Group Card

041/VER-01



A (AB01)	B (AB02)	AB (AB03)	D (RH1)	Ctrl Neutral	AHG